VITALITY

BLOOD PRESSURE MONITOR

MANUAL

ECB002

1282

Xinyu Electronic Co.,Ltd
No.3, Futian 5 Rd, Shite Industrial District,
Gangkou Town, Zhongshan, Guangdong, PRC

BLOOD PRESSURE MONITOR WITH ARM CUFF

WHAT IS BLOOD PRESSURE?


The blood pressure is the pressure the blood is exerting on the walls of the arteries. The blood pressure in the arteries is subject to continuous variation during a heart cycle. The highest pressure within this cycle is called 'systolic pressure' (SYS), the lowest pressure is called 'diastolic pressure' (DIA). With these two blood pressure values, a doctor can determine a patient's blood pressure. Your blood pressure may be influenced by many different factors as physical effort, anxiety or the time of the day etc. Generally, blood pressure is low in the morning and raises in the afternoon towards the evening. In summer, blood pressure is lower than in winter.

WHY IS IT A GOOD THING TO MEASURE BLOOD PRESSURE AT HOME

Having one's blood pressure measured by a doctor in a hospital or clinic environment tends to stimulate nervousness in a patient and may lead to a higher blood pressure. Blood pressure varies in accordance with a variety of conditions and so judgment is not always possible on the basis of a single measurement. Blood pressure measured first thing in the morning after getting up, before eating, and keeping still, is known as the fundamental blood pressure. In practice it is rather difficult to record the fundamental blood pressure but measurement at home in the right environment aids this.


BLOOD PRESSURE CLASSIFICATION OF THE WHO

The standards for the determination of high or low blood pressure independently from the patient's age – as shown on the table – were determined by the World Health Organization (WHO).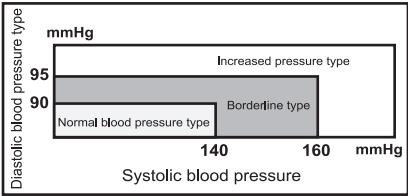

VARIATIONS OF BLOOD PRESSURE

Individual blood pressure is subject to variations on a basis or in relation to the time. These variations are more perceptible in patients with high blood pressure. Usually, blood pressure is higher during work and is lowest in the night when sleeping. The figure shows the variations of the blood pressure in the course of a day with measurements taken every 5 minutes. The bold line represents the sleep phase. The increase in blood pressure at 4 pm (shown as letter 'A' in the figure) and at midnight (shown as letter 'B' in the figure) mark pain events of the patient.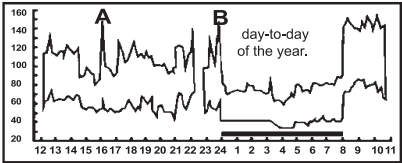

TECHNICAL SPECIFICATIONS

Measuring method	Oscillometric
Memory capacity	90 records
Display	LCD display size 90 * 55mm
Measuring range	Pressure: 0—300 mmHg (0-40 kPa) Pulse: 40 – 199 pulses/minute
Measurement accuracy	Pressure: ± 3 mmHg (0,4 kPa) at an ambient temperature of 15-25 °C ± 6 mmHg (0,8 kPa) at an ambient temperature of 10-14 °C and 26-40°C, Pulse: $\pm 5\%$
Adjustable length of the cuff	22-32 cm

IMPORTANT INSTRUCTIONS OF THE DEVICE

- Blood pressure should be judged by a medical doctor or other medical specialist who is familiar with your case history. Through the regular use of the blood pressure gauge you can continuously record your pressure values. These records can be used for therapeutic purposes.
- Tightly wrap the cuff around your upper arm. The cuff must be **positioned on the level of the heart**.
- The monitor must not be moved or shaken during measuring as this would lead to inaccurate results.
- Carry out measuring in a still and relaxed position.
- Do not wrap the cuff around the sleeve of a jacket, shirt which could make measuring impossible.
- Keep in mind that blood pressure naturally varies in the course of the day and is influenced by many different factors as, e.g., smoking, alcohol, drugs and physical effort. We recommend not to smoke, eat or do physical exercises for about 30 minutes before measuring.
- To the people with following condition which leads to circulatory disorders (diabetes, kidney diseases, hardening of the arteries), the values measured with this device may be lower than those measured with a gauge taking the blood pressure at the upper arm. Consult your doctor to make sure that measuring blood pressure at your upper arm leads to accurate measuring values.
- The measuring device is designed for the measuring of blood pressure in adults only.
- Do not use the device if your upper arm is injured.
- If the automatic inflation function of the cuff does not stop automatically it must be opened and taken off immediately!

Note User group: This device is not suitable for invasive blood pressure measuring in adults or children. It may only be used for non-invasive blood pressure monitoring and only in adults!

SELECTING THE POWER SOURCE

A. Putting in the batteries

Remove the cover of the battery compartment on the lower side of the measuring device and put the batteries into the battery compartment as the signs showed inside the battery compartment. Put in 4 batteries of type 1.5V, type AAA. Pay particular attention to the correct positions of the + and – poles of the batteries. Close the battery compartment.

B. Operation with mains adaptor

You can optionally operate the device over a mains adaptor (not included).

- Put the plug of the mains adaptor into a properly installed shockproof socket (230V, 50Hz).
- Connect the device to the main adaptor by putting the plug of the mains adaptor into the 6V DC jack of the device.
- When using a mains adaptor (6V DC) take care to apply the correct polarization.

MEASURING INSTRUCTIONS

Set the date, time and measurement unit


- Before taking measurements, set the current date, time and measurement unit. The measured value will be stored in memory together with the date and time of measurement.
- Check that the device is turned off (the LCD display is off) or turn it off using the  button.
- Press the button SET, and hold for 3 seconds, the default year will flash on the display. During the initial setup the display will show year (2012), month (1) and day (1).

- Set the current year by pressing the button MEM and confirm it by pressing button SET. And the display show another page with date and time.
- The first two numbers (which means month) are flashing. Press the button MEM to put the right month and confirm by pressing button SET.
- The date numbers flash. Press the button MEM to put the right date and confirm by pressing button SET.
- The hour number flash and press the button MEM to put the right hour, This device is set with 12-hour time format. When the time is over 12, the LCD shows PM. And confirm the hour with button SET.
- The minute numbers flash. Press button MEM to put the right minute and confirm by pressing button SET.
- The LCD shows another page for unit selection, and the letter "ON" flashes. Press MEM to select "off" or "on" function to select the measurement unit. Select OFF function, which means measurement unit will be mmHg. Select ON function, which means the measurement unit will be kPa, but the measurement result DIA will be shown both in kPa and mmHg, and the two results will show in turn. Confirm the setting by pressing button SET. And the device automatically turns off.

MEASURING BLOOD PRESSURE

Basic instruction for achieving the most accurate measuring results

- Always take measurements at the same time of day, ideally in the morning, at noon and in the evening under the same conditions or according to the recommendations of your doctor.
- Do not perform measurement sooner than 30-45 minutes after consuming coffee, tea or smoking a cigarette.
- Wait at least 20 minutes after taking a hot shower or bath.
- During measurement sit calmly, relax and don't talk. Do not move the arm to which the cuff is attached.
- Wait approximately 4-5 minutes before measuring again.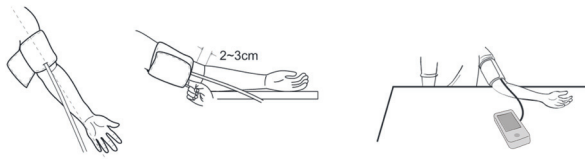

Wrap the cuff around the upper arm

- Open the Velcro fastener and pull to open the cuff.
- Lead your arm through the cuff and pull the cuff up to a position 1-2 cm above your elbow. The cuff is suitable for a perimeter of the upper arm about 22-32 cm.
- Pull the cuff tight around the upper arm. Do not pull on the cuff with exaggerated force.
- Do not pull the cuff too tight, leave a gap of about the width of a finger.

Correct body position for measuring

To receive best possible measuring results make sure that the cuff is held on the same level as your heart. If the cuff is held in a position above or below the heart this may lead to deviations from the exact value.

MEASURING BLOOD PRESSURE IN SITTING POSITION

- Rest your elbow on a table or other surface (as, e.g., a bag).
- Position your arm on the rest in a way that makes sure that the cuff is on the same level with your heart.
- Relax your arm. The palm of your hand must point upwards.


MEASURE THE BLOOD PRESSURE

- Put the device on a solid and level surface.
- Connect the cuff, i.e., the rubber hose to the connection.
- Wrap the cuff around your upper arm.
- Sit down on a chair and take up an upright position.

• Push the  button and hold it down for about 1 second. The number '888' will be shown on the display for 3 seconds. Then, the display will show the value '0'. Air pressure is automatically increased to 190 mmHg. In case of a tendency to high blood pressure, the air pressure is automatically increased to the required level (up to 300 mmHg). Do not move or talk during the measuring cycle.

• After the air pressure has been increased it is slowly let out. As soon as the pulse has been detected, the symbol on the display starts to flash.


• After the measuring cycle has been completed, the values of the systolic, diastolic pressures and the pulse rate are shown on the display. To finish the measurement, push the  button and LCD displays SURE. If you do not turn off the device, it will turn off automatically 1 minute after the last measurement. Remove the cuff from your arm after completing the measurement.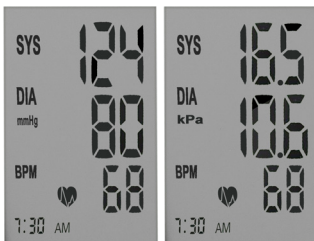

IRREGULAR HEARTBEAT

During the measurement, the heartbeat symbol  will flash. When the test finished, LCD display will show up the heartbeat rate and this symbol will disappear. When the device detects an irregular heartbeat rhythm two or more time, the heartbeat symbol will stop flashing and show up together with the heartbeat rate.

An irregular heartbeat rhythm is defined as a rhythm that varies by 25% higher or lower than average rhythm detected while the device is measuring the systolic and diastolic blood pressure. When you are detected with irregular heartbeat rhythm, please consult with your doctor.

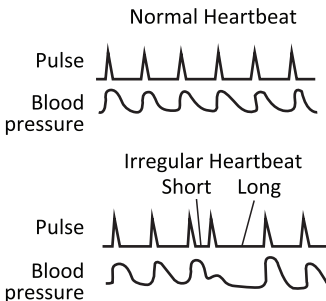SAVE THE MEASURING DATA AND SHOW THE MEMORY

a. Save data

After each measurement of the blood pressure measurement, the data of the systolic/diastolic pressures and heart rate of the respective measuring cycle are automatically saved. The last 90 measuring results can be retrieved from the device's memory. If the number of measuring cycles exceeds 90, the oldest data in the memory are automatically deleted.

b. Display the stored measurement results.

To display the stored measurement results press the button MEM. The display will show the stored measurement results. Press again the MEM button to display more results from earlier measurement results.

MAINTENANCE

- To clean the housing of the device use a cloth moistened with water or a soft cleaning agent, then dry it with a dry cloth. Use a dry cloth to clean the cuff if it should be dirty.
- Do not immerse the device in water.
- Do not use gas or strong cleaning agents.
- If the measuring device is not used for a longer time, the batteries should be removed. (Leaking batteries may damage the device.)

STORING IN THE BAG

- Always store the cuff in the provided bag after use.
- Protect the measuring device with its accessories from direct sunlight, high temperatures, humidity and dust.
- Do not expose the device to extremely low (below -20°C (-4.0°F)) or high (above 65°C (122.0°F)) temperatures.

Error code	problems	check
LO	1)Low power 2) abnormality of gas circuit.	1)Change the batteries, we suggest to use 1.5V alkaline battery. 2)Tighten the cuff.
Er 0	LCD display didn't show '0" in 10 second	Data loss, recalibrate device.
Er2	1)Weak signals, or loosely cuff. 2)Cuff is pressed by something during testing. 3)Plug of cuff didn't insert port of device correctly. 4)Cough during test. 5)User move or move device during testing.	1)Tighten the cuff, 2) Don't press cuff during testing. 3) Insert plug of cuff to port of device correctly. 4) Keep calm during testing 5) Don't move or keep device at the same place during testing.
Er4	No rating data.(rare error)	Recalibrate device.
Er10	Blood pressure exceeds 300 mmHg or design limit for protection pressure. (rare error)	Check if user wear something on wrist inside cuff.
Er30	After turning on device, inflated value is lower than standard in prescribed time (10 s)	Cuff is not worn during testing.

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WARRANTY

1. This warranty is issued by Esperanza sp.j. Poterek with address:
ul. Poznańska 129/133, 05-850 Ożarów Mazowiecki, Phone: 022-7213571 w.10 – called hereinafter Esperanza.
2. This warranty confers upon you specific rights and remedies against Esperanza. Please read this document carefully, by making a claim under this warranty, you will be deemed to have understood and accept its terms.
3. Esperanza warrants that any Esperanza product together with attached accessories is free from defects in materials and workmanship under conditions of its normal use.
4. If you make a claim under this warranty, Esperanza will, at its option, repair or replace any defective product into new one by using new or use factory-refurbished parts. In case of using factory-refurbished parts they will be equal to new products.
5. Defective product should be returned together with receipt, commercial invoice or any other proof of purchase to the place of purchase. It should include all accessories attached in original packing. Esperanza has right to reject warranty if any accessories will be missed.
6. Esperanza is committed to respond to the complaint within 14 calendar days of its notification in the point of sale. Esperanza will repair the defective product in the shortest possible time. If will be need to import from abroad of any parts which are necessary to repair this time may be extended.
7. In case of replacing defective product onto new one, Esperanza has right to replace it with newer model with the same technical parameters if the old one model was replaced by new one during warranty period.
8. If you make claim under this warranty, you are deemed to have agreed that repair and (upon availability) replacement, as applicable, under the warranty services described herein, is your sole and exclusive remedy vis-a-vis Esperanza.

EXCLUSIONS:

This warranty does not extend to:

1. Any product not manufactured by Esperanza
2. Any product that has been damaged or rendered defective as a result of:
 - use it in other way than normal intended use
 - modification or internal interferes or changes of product
 - service or repair by anyone other than Esperanza service
 - improper store, transportation or packing
 - improper installation of product or software
 - mechanical, physical, chemical etc. damages
3. In no event shall Esperanza be liable for any damages, loss of business, profits, data or loss based on use Esperanza products with other devices.
4. Products with removed warranty and other labels and products spent or have hard use marks.

All products are covered by Esperanza's 2-year manufacturer's warranty while product is purchased by the consumer (the consumer is a person who buys product for the purpose of not professional or commercial activity). In the case of purchase for the company (for purposes relating to the activities of professional or commercial) warranty is 1 year from date of purchase. Esperanza products have all the necessary safety certificates required in the EU.

According to Directive 2002/96/EC, disposal of waste electrical and electronic equipment together with household waste is prohibited. Their components need to be recycled or disposed separately. Otherwise, they can cause the release of hazardous and noxious substances which pose a threat to the environment. The law requires each consumer to return unnecessary or waste electrical and electronic equipment to the producers, sales agents or waste treatment facilities. Detailed conditions are governed by national legislation. The above symbol placed on the product, in the user manual or on the packaging refers to this requirement. Thanks to proper segregation and compliance with the provisions relating to waste disposal each user contributes significantly to environmental protection.

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Certificate of Compliance

No. 0P160826.XETQ25

Technical Construction File no. TMGD1608108662

Certificate's
Holder:

Xinyu Electronic Co., Ltd.
No.3,Futian 5 Rd, Shite Industrial District, Gangkou
Town, Zhongshan, Guangdong, PRC, Postal Code:
528447

Certification ECM
Mark:

Product:
Model(s):

Electronic Blood Pressure Monitor
XY-B01, XY-B02, XY-B03, XY-B05, XY-B06,
XY-W01A, XY-W01B.

Verification to:

Standard:
EN 60601-1-2:2007+AC:2010,
EN 60601-1-11:2010, EN 60601-2-30:2010,
EN 1060-3:1997+A2:2009

related to CE Directive(s):
93/42/EEC amended by 2007/47/EC (Medical
Devices)

Remark: The product(s) has been verified on a voluntary basis. The product(s) satisfies the requirements of the Certification Mark of ECM, in reference to the above listed Standard(s). The above Compliance Mark can be affixed on the product(s) accordingly to the ECM regulation about its release and its use. The regulation can be found at www.entecerma.it. This Certificate of Compliance can be checked for validity at www.entecerma.it.

This verification doesn't imply assessment of the production of the product(s).

Additional information, clarification about the **CE** Marking:

We attest that a TCF for the **CE** Marking process is in place. Whereas the Manufacturer is Responsible to start the **CE** Marking Certification Procedure through an appointed Notified Body and the perform all the necessary activities, as required by the Directive and accepted by the Notified Body, before placing the **CE** Mark on the product(s).

Date of issue 26 August 2016

Expiry date 25 August 2021

Chief Manager
Tim Mahan

Deputy Manager
Viola Miller

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Via Ca' Bella, 243 – Loc. Castello di Serravalle – 40053 Valsamoggia (BO) - ITALY
☎ +39 051 6705141 ☎ +39 051 6705156 ✉ info@entecerma.it 🌐 www.entecerma.it

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Ente Certificazione Macchine Srl